# INCREDIWEAR

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2020 Incrediwear Holdings, Inc  
Datum izdaje navodil: 20.01.2021

# **i**NCREDIWEAR

WEARABLE  
**ANTI-INFLAMMATORY**  
THERAPY

## **INSTRUCTIONS FOR USE AND MAINTENANCE**

### **Manufacturer:**

Incrediwear Holdings, Inc.  
3120Thorntree Drive,  
Chico, CA, 95973

[www.incrediwear.com](http://www.incrediwear.com)

### **Importer for Slovenia:**

ONNEA d.o.o.  
Vransko 184,  
3305 Vransko

[www.incrediwear.si](http://www.incrediwear.si)

### **HOW DOES INCREDIWEAR WORK?**

Incrediwear's revolutionary fabric touches your skin, your body heat activates the semiconductor elements leading to greater blood flow and faster circulation.

This increased blood flow means more oxygen is supplied to the area, thereby helping to reduce swelling, pain, fatigue, and stiffness.

Incrediwear products are created by medical specialists, and the increased blood flow and circulation speeds have been proven using laser flowmetry tests.

World renowned physicians recommend Incrediwear to aid recovery after injury or surgery, to prevent injury, and to accelerate recovery during and after certain activities.

### **INSTRUCTIONS FOR USE**

**AFTER AN INJURY:** Can be used 24 hours per day until swelling and pain are reduced.

**FOR PAIN AND INFLAMMATION:** Can be used 24 hours per day or as necessary for eventual pain or inflammation.

**FOR INJURY RECOVERY AND PREVENTION:** Can be used 24 hours per day or before, during, and after certain activities.

**FOR ACTIVE RECOVERY:** Use during physical activity and for a minimum period of 2 hours after exercise.

All Incrediwear products can be used while sleeping, this further helps to recover after activities, heal injuries, and reduce pain and swelling.

If the pain persists, please consult your doctor.

### **WASHIN INSTRUCTIONS**

Incrediwear products can be hand-wash in cold water and tumble dried at low temperatures.

Products with Velcro (bandages and back, hip and shoulder braces) should be placed in a suitable bag before being placed in a washing machine.

Use cold water and line dry, or hand wash in cold water and line dry.

Do not iron or dry clean.

No special storage instructions need to be followed.

Obelis s.a.  
Boulevard General Wahis 53  
1030 Brussels, BELGIUM  
You can find the Declaration of  
Conformity on  
[www.incrediwear.si](http://www.incrediwear.si)

Designed in Chico, California  
Incrediwear Holdings, Inc.  
3120 Thorntree Drive  
Chico, CA 95973, USA

2020 Incrediwear Holdings, Inc  
Date of issue: 20.01.2021